


Extra Care

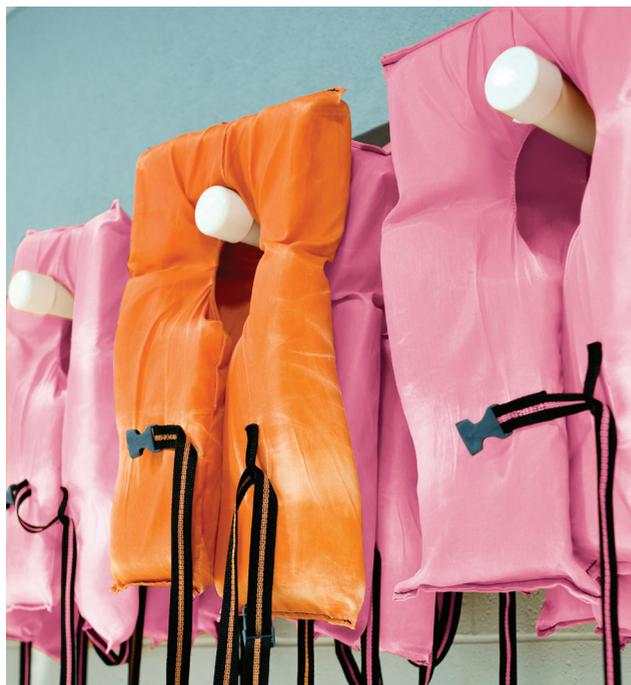

Dansac Accessories

Introduction

At Dansac, we are dedicated to developing unique ostomy pouches and skin barriers to help improve the quality of life for people with a stoma.

Individual differences may, however, lead to a need for extra care products.

In addition to offering ostomy pouch systems, we also care for these particular ostomy needs by offering easy-to-use extra care products that can make everyday life more comfortable and secure.

This folder describes the different products that Dansac currently offers. These products may help ostomates who are experiencing problems with leakage, skin dryness, skin irritation, unpleasant odours etc.

We hope you will find this folder informative and helpful.

Dansac A/S

Contents



Dansac Seal..... 4

Dansac GX-tra Seal 6

Dansac Soft Paste..... 8

Dansac Adhesive Remover 10

Dansac Skin Protector..... 12

Dansac Skin Lotion 14

Dansac Skin Creme 16

Dansac Nodor S..... 18

Notes..... 20



DANSAC SEAL

Extra protection where needed the most

Extra protection where needed the most

Skin folds, scars and retractions can make it difficult for your patients to get a good seal between the skin and the appliance. The unique design of the Dansac Seal with an off centered starter hole is ideal for positioning the seal where extra protection is needed. Dansac Seal can be applied to the skin or to the barrier and is easily removed without leaving residue on the skin.

Due to the shape of the seals, where the barrier material is unevenly distributed around the center hole, it is possible to turn the seals in the direction where extra protection is needed. With 4 sizes to choose from, there is a Dansac Seal for every stoma.

Dansac Seal is suitable for all stomas and is made of a non-alcohol skin barrier material with microscopic reinforcing fibers, the 3 mm seal is easy to stretch and can be shaped again and again if needed without breaking or losing adhesive ability.

The Dansac Seals are individually packaged. This provides a simple and user-friendly solution with snap lock which makes it easy to open. The biodegradable cover tray makes it convenient for storage and everyday use.



DANSAC SEAL
Accessories



Ring size	REF	Pcs
20 mm	070 - 20	30
30 mm	070 - 30	30
40 mm	070 - 40	30
50 mm	070 - 50	30



- Extra protection
- Off centered starter hole



DANSAC GX-tra **SEAL**

Extra security against leakage

Extra security against leakage

Do your patients need extra security or worry about leakage? Dansac GX-tra Seals could be the ideal solution. In addition to providing security against leakage, GX-tra Seals also increase the wear time of the pouch.

GX-tra Seals are ideal for flush or retracted stomas to help build up the area under the skin barrier and protect the skin against corrosive effluent.

Stretch the GX-tra Seal to fit around the stoma or cut it up into smaller pieces to fill folds and contours. When an even surface has been created, just attach the ostomy appliance as usual. You can also attach the seal directly onto the skin barrier.

GX-tra Seals are made from a hypoallergenic GX hydrocolloid with cotton fibres. The cotton fibres help increase wear time by containing the moisture and slowing down the dissolving of the hydrocolloid. The seals do not contain alcohol and are therefore ideal to protect sensitive skin and promote a safe healing environment.




DANSAC GX-tra SEAL
Accessories



Ring size	REF	Pcs
20 mm	725 - 20/30	30
30 mm	725 - 30/30	30
40 mm	725 - 40/30	30
50 mm	725 - 50/30	30



- Security against leakage
- Longer wear time



DANSAC SOFT **PASTE**

Levelling skin folds and scars

Levelling skin folds and scars

It is not unusual to have uneven skin or skin folds immediately around the stoma. These conditions, however, can make it difficult to obtain a secure and close fit of the ostomy appliance.

Dansac Soft Paste is ideal for levelling skin folds, scars and wrinkles to make the appliance fit securely around stomas or fistulae. The paste helps protect the skin by reducing the risk of effluent tracking under the wafer. It also extends the wear time of the appliance.

The paste is easily applied to dry skin or directly onto the skin barrier using a moistened finger or spatula to shape and spread the paste and obtain a smooth surface. Let it dry for 30 seconds before further application. This will evaporate the alcohol and reduce stinging on broken peristomal skin.

If necessary, Dansac Soft Paste can be used with ostomy seals like Dansac GXtra Seal or Dansac Seal.



DANSAC SOFT PASTE
Accessories

Volume	REF
14.2 g	77514 - 2
50.0 g	77550 - 0

- Helps level the skin around the stoma
- Protects the skin



DANSAC ADHESIVE **REMOVER**

Quick and easy removal of appliance

Quick and easy removal of appliance

The Dansac Adhesive Remover promotes quick and painless removal of stoma appliances.

Because the Dansac Adhesive Remover is a silicone based alcohol-free solution it will not sting, even on sore skin. It dissolves excess adhesive facilitating fast and easy removal of the pouch.

Dansac Adhesive Remover is available in single and easy-to-use wipes and as a 50 ml spray.

The products can be used separately or in combination: use the spray to remove the barrier and a wipe for any leftover residue. The wipes are discreet and practical to bring along in a bag or pocket.



DANSAC ADHESIVE REMOVER
Accessories

	REF	Pcs/Volume
Wipes	081 - 30	30 pcs
Aerosol spray	082 - 01	50 ml

- No stinging
- Fast and easy removal of appliance



DANSAC SKIN **PROTECTOR**

Extra protection for your skin

Extra protection for your skin

Effective protection against peristomal skin irritation and adhesive trauma is vital in maintaining skin integrity. Dansac Skin Protector is the ideal choice to maintain this protection and increase comfort.

Dansac Skin Protector is a silicone based alcohol-free protective film that helps gently protect the skin against stoma output and stoma pouch adhesives.

Dansac Skin Protector is available in single and easy to use, non-sting wipes.



DANSAC SKIN PROTECTOR
Accessories

	REF	Pcs
Wipes	081 - 30	30 pcs

- No stinging
- Helps prevent sore skin



DANSAC SKIN LOTION

Efficient cleansing of peristomal skin

Thorough cleaning of your peristomal skin

In general, pure water is recommended for cleaning of the peristomal skin. Sometimes, however, there is no access to clean water - and sometimes the skin needs a little extra. Dansac Skin Lotion is a gentle soap, used and trusted by ostomates for over 20 years. Ideal for cleaning the skin around the stoma.

Apply Dansac Skin Lotion to a soft tissue and use this tissue to clean and soothe the skin around the stoma. After application, dab the skin gently before applying the next pouch.

Dansac Skin Lotion is available in a 200 ml bottle, a 50 ml bottle, and as wet tissues.



DANSAC SKIN LOTION
Accessories

Volume	REF	Pcs
50 ml	70005 - 0001	
200 ml	70000 - 0000	
Tissues	71000 - 0000	50



- Cleans the skin around the stoma
- No need for water



DANSAC SKIN **CREME**

Extra moisture for your skin

Avoid dryness and skin irritations

Even with the most advanced, skin barriers, it is natural that the skin around the stoma can be dry from time to time. By applying Dansac Skin Creme to the skin before applying the skin barrier, possible skin irritation and dryness may be avoided.

The creme contains Aloe Vera and vitamins A, D and E and has a soothing and moisturizing effect on the skin. Since Dansac Skin Creme is non-greasy, it does not affect the adhesion of the skin barrier. The creme has no colour or scent.

The creme is quickly absorbed and leaves the skin smooth and soft - ready for a new ostomy appliance.

Dansac Skin Creme is available in a 50 ml and a 100 ml bottle with a practical and convenient applicator.



DANSAC SKIN CREME
Accessories

Volume	REF
50 ml	085 - 50
100 ml	085 - 00

- Soothes and moisturizes the skin
- Helps avoid skin irritation



DANSAC **NODOR S**

Odour prevention for extra discretion

Prevents unpleasant odours

Adding a few drops of Nodor S into the pouch helps neutralize odours from faeces, and the risk of unpleasant smells when changing or emptying the pouch is minimized.

Dansac Nodor S is a safe and harmless product, consisting of water and a solution containing aluminium and zinc salts. It works by absorbing odours through the salt molecules and it has no fragrance in itself.

To use Dansac Nodor S, pour a few drops into the pouch. Depending on digestion and diet, it may be necessary to add a few extra drops to obtain the full effect.

Dansac Nodor S is available in two sizes - the small 50 ml bottle is convenient to carry around with you, the larger 250 ml bottle is for use at home.



DANSAC NODOR S
Accessories

Volume	REF
50 ml	080 - 00
250 ml	080 - 01

- Avoid unpleasant odours

Dansac Accessories

Sometimes the skin needs a little extra help or protection
Dansac offers a variety of accessory products to help take care of your skin:

Dansac Seal

Extra protection where needed the most

Dansac GX-tra Seal

Extra security against leakage

Dansac Soft Paste

Levelling skin folds and scars

Dansac Adhesive Remover

Quick and easy removal of appliance

Dansac Skin Protector

Extra protection for your skin

Dansac Skin Lotion

Efficient cleansing of peristomal skin

Dansac Skin Creme

Extra moisture for your skin

Dansac Nodor S

Odour prevention for extra discretion

P02-60-300 02/13 © Dansac Lindegaard & Co.